

YELLOW PORCELAIN ETCH

Porcelain etch



BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY
PRODUCT FOR DENTAL USE ONLY

COMPOSITION: hydrofluoric acid

INTENDED USE

Product is used for porcelain etching during reparations of the dentures and as well as for the etching of the porcelain veneers, crowns or inlays before their cementation.

INSTRUCTIONS FOR USE

1. Isolate operation field with rubber-dam.
2. Screw elastic blue applicator on the syringe with the etchant, replace safety plug with this applicator.
3. Apply small amount of etching preparation on the prepared and dried surface. After use screw the syringe with a safety plug.
4. Leave etching for 60 seconds and then rinse and dry the procedure spot.
5. Apply Silan according to its instruction for use provided by producer.

CONTRAINDICATIONS

Do not use the product for patients with hypersensitivity to the product's components.

CAUTIONS

Corrosive preparations – avoid contact with skin and mucosa membrane, in case of contact rinse immediately with copious amount of running water, seek medical advice.

Avoid contact with eyes, in case of contact rinse with plenty of water with wide open lid, seek ophthalmologist advice. In case of consumption do not extract the emesis, seek medical advice.

In case of any reparations of the dentures directly in the patient's oral cavity it is required to use rubber-dam and any protection for patient's eyes. Use vacuum system while etching. Equip personel with protective clothing, gloves and visors.

STORAGE

Store in original packaging at temperature below 25 °C.

Store out of reach of children.

Expiry date is placed on the main container.

Once container is open, the expiry date doesn't change if it is securely closed after each use.

HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly.

PACKAGING

Syringe of 2ml of preparation, disposal elastic applicators.



PPH CERKAMED Wojciech Pawłowski
37-450 Stalowa Wola, Polska
ul. Kwiatkowskiego 1
Phone/fax 0048 15 842-35-85