Prior to use, carefully read the instructions for use.

EN

## GC Fuji™ I

RADIOPAQUE GLASS IONOMER LUTING CEMENT

For use only by a dental professional in the recommended indications.

### RECOMMENDED INDICATIONS

- Cementation of metal-based inlays, onlays, crowns, bridges and posts.
- Cementation of high strength (zirconia based) all ceramic crowns and bridges.

### CONTRAINDICATIONS

- Pulp capping.
- In rare cases the product may cause sensitivity in some people. If any of such reactions are experienced, discontinue the use of the product and refer to a physician.

### DIRECTIONS FOR USE

|                                                                 |         |
|-----------------------------------------------------------------|---------|
| Powder/liquid ratio (g/g)                                       | 1.8/1.0 |
| Mixing time (min., sec.)                                        | 20"     |
| Working time (min., sec.) (at 23°C) (from start of mixing)      | 2'00"   |
| Final finishing commencing time (after seating the restoration) | 4'30"   |

- TOOTH PREPARATION
  - Prepare tooth in usual manner. For pulp capping, use calcium hydroxide.
  - Clean the prepared teeth with pumice and water.
  - Rinse thoroughly with water. Remove excess moisture by blotting with a cotton pellet or gently blowing with an oil-free air. DO NOT DESICCATE. Prepared surfaces should appear moist (glistening).

- RESTORATION PREPARATION

Make sure that the restoration is pretreated and handled according to the manufacturer's instructions.

- POWDER AND LIQUID DISPENSING
  - The standard powder to liquid ratio is 1.8g/1.0g (1 level scoop of powder to 2 drops of liquid).
  - For accurate dispensing of powder, lightly tap the bottle against the hand. Do not shake or invert.
  - Hold the liquid bottle vertically and squeeze gently.
  - Close bottles immediately after use.
- MIXING

Dispense powder and liquid onto the pad. Using the plastic spatula, add all the powder to the liquid. Mix rapidly for 20 seconds.

Note :

When mixing larger amounts, divide the powder into two equal parts. Mix the first portion with all of liquid for 5 seconds. Incorporate the remaining portion and mix the whole thoroughly for 15 seconds (total 20 seconds).

- CEMENTATION
  - Coat the internal surface of the restoration with sufficient cement and seal immediately. The working time is 2 minutes from start of mixing at 23°C (73,4°F). Higher temperatures shorten working time.
  - Maintain moderate pressure.
  - Start removing excess cement when the excess cement feels rubbery.
  - Finishing can be started 4 minutes 30 seconds after seating the restoration.

### STORAGE

Store in a cool and dark place (4-25°C) (39,2-77,0°F). (Shelf life : 3 years from date of manufacture)

### SHADE

Light yellow

### PACKAGES

- 1-1 package : 35g powder, 25g (20mL) liquid, powder scoop, mixing pads (No. 20).
- Bottle of 35g powder with scoop.
- Bottle of 25g (20mL) liquid.

### CAUTION

- In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water. To avoid contact, a rubber dam and/or cocoa butter can be used to isolate the operation field from the skin or oral tissue.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- DO NOT mix powder or liquid with components of other glass ionomer cements.
- This product is not indicated for filling or core build-up.

Last revised : 09/2014

CE 0086

**GC**

MANUFACTURED BY:  
GC CORPORATION  
75-1 Hasunuma-cho, Ibaashi-ku, Tokyo 174-8585, Japan

DISTRIBUTED BY:  
GC CORPORATION  
75-1 Hasunuma-cho, Ibaashi-ku, Tokyo 174-8585, Japan

EU: GC EUROPE N.V.  
Plessesteigpark Nazdorde-Leuven 1240, Interleuvenlaan 33, B-3001 Leuven, Belgium  
TEL: +32 16 74 10 00

GC AMERICA INC.  
3737 West 127th Street, Atlap, IL 60803 U.S.A.  
TEL: +1-708-597-0900

GC ASIA DENTAL PTE. LTD.  
11 Tampines Concourse, #03-05, Singapore 528729  
TEL: +65 6546 7588

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